



ARJO STANDARD 4 POINT PADDED SLING WITH HEAD SUPPORT



SKU: MAA4000M-H

PRODUCT DESCRIPTION

PURPOSE & BENEFITS

General Purpose Clip Sling with Padded Legs With padded legs for extra comfort, the general purpose sling is used for a range of everyday patient handling routines, such as bed and chair-related transfers. A padded solution is best for residents/patients who would benefit from a little extra comfort. Padding improves comfort in the sling's leg area.

This General Purpose Sling is easy to use and will be suitable for a variety of lifting hoists, and is available in a variety of materials (see the Technical tab).

Place the back of the General Purpose Sling behind the patient's back so the bottom of the Sling is next to the coccyx area. The stitches in the middle of the Sling should be parallel with the patient's spine.

The leg sections should go under the patient's legs. Thread the webbing from one leg section through the loop on the opposite leg webbing.

FEATURES

- General use Sling
- Clip sling for Arjo lifts
- Padded legs
- U-sling style

SPECIFICATIONS

- Safe Working Load - 272 kg/600 lbs

For further information go to **medimart.com.au** or give us a call: **1300 633 063**

SA

Brighton Showroom

343 Brighton Rd, North
Brighton SA 5048
08 7079 8773

brighton@medimart.com.au

Elizabeth Vale Showroom

Playford Health Hub, 44 John Rice
Ave, Elizabeth Vale SA 5112
08 7079 8777

elizabethvale@medimart.com.au

Reynella Showroom

Shop 1/141 Sherriffs Rd,
Reynella SA 5161
08 7079 8771

reynella@medimart.com.au

VIC

Frankston Showroom

111 Cranbourne Rd, Frankston VIC 3199
03 8763 3451

frankston@medimart.com.au

Monash Showroom

23 Winterton Road, Clayton VIC 3168
03 7503 8354

monash@medimart.com.au

NT

Darwin Showroom

Unit 4/524 Stuart Hwy, Winnellie NT 0820
1300 633 468

support@medimarnt.com.au