



## ARM SLING OPPO MESH LARGE

**\$16.00** Online Only



**SKU:** OPP308750

## PRODUCT DESCRIPTION

### PURPOSE AND BENEFITS

The OPPO Mesh Arm Sling designed with a breathable mesh netting for ultimate comfort aiding in keeping injured shoulders or arms in neutral positioning. This product is ideal for bronchial nerve injury, upper-arm fracture, shoulder subluxation or dislocation and post-surgery protection,

Equipped with height adjustable padded straps and a thumb loop to aid in keeping the hand correctly positioned.

This product is universally designed to fit both the left and right arm.

### FEATURES

- Offers immobilisation and support for mild sprains/strains of the shoulder area.
- Netting material for greater comfort, particularly for long-term use.
- Helps keep shoulder and arm in a neutral position.
- Adjustable padded straps for height adjustment. Thumb loop helps keep hand correctly positioned.

### HOW TO WEAR

- Insert the forearm into the sling.
- Place thumb in the thumb strap of the sling.
- Pull strap over shoulder and Insert the hook end of the strap into the buckle.
- Adjust the length of the shoulder strap so the elbow joint rests at an approximate 90° angle.
- Fasten hook and loop closure to tighten.

**SIZING CHART: MEASURE LENGTH OF FOREARM**

S	23.5-25.4cm
M	25.4-30.5cm
L	30.5-35.6cm
XL	35.6-40.6cm