

## ETAC SOCKY STOCKING AID



**SKU:** 8060100-H

## PRODUCT DESCRIPTION

### PURPOSE AND BENEFITS

The Etac Socky stocking aid is specifically designed to assist with putting on support and compression stockings. It accommodates various lengths and types of stockings, making it versatile and suitable for different needs.

The Socky consists of two long sections of nylon fabric that connect with Velcro. Each section is equipped with a large handle, providing a secure grip for pulling the stocking over the foot and leg with ease.

Using the Socky for support and compression stockings is simple. Start by fastening the Velcro and folding the nylon fabric. Then, slide the stocking over the Socky. Grip the handles with your hands and pull up the stocking by twisting the Socky around your hands until it splits into two parts.

The Etac Socky stocking aid offers a practical solution for individuals needing assistance in putting on support and compression stockings. Its user-friendly design simplifies the process and promotes independence and convenience.

### FEATURES

- Designed for putting on support and compression stockings
- Accommodates different lengths and types of stockings
- Consists of two long nylon fabric sections connected with Velcro
- Each section features a large handle for a secure grip
- Fasten the Velcro and fold the nylon fabric
- Split into two parts for easy removal

- Assists individuals in independently putting on support and compression stockings

For further information go to **medimart.com.au** or give us a call: **1300 633 063**

## SA

### **Brighton Showroom**

343 Brighton Rd, North  
Brighton SA 5048  
08 7079 8773

[brighton@medimart.com.au](mailto:brighton@medimart.com.au)

### **Elizabeth Vale Showroom**

Playford Health Hub, 44 John Rice  
Ave, Elizabeth Vale SA 5112  
08 7079 8777

[elizabethvale@medimart.com.au](mailto:elizabethvale@medimart.com.au)

### **Reynella Showroom**

Shop 1/141 Sherriiffs Rd,  
Reynella SA 5161  
08 7079 8771

[reynella@medimart.com.au](mailto:reynella@medimart.com.au)

## VIC

### **Frankston Showroom**

111 Cranbourne Rd, Frankston VIC 3199  
03 8763 3451

[frankston@medimart.com.au](mailto:frankston@medimart.com.au)

### **Monash Showroom**

23 Winterton Road, Clayton VIC 3168  
03 7503 8354

[monash@medimart.com.au](mailto:monash@medimart.com.au)

## NT

### **Darwin Showroom**

Unit 4/524 Stuart Hwy, Winnellie NT 0820  
1300 633 468

[support@medimarnt.com.au](mailto:support@medimarnt.com.au)