




OPPO POSTURE AID / CLAVICLE BRACE

\$59.00 - \$78.00 Price range: \$59.00 through \$78.00

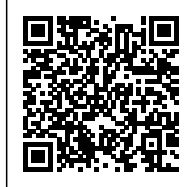

SKU: Opp2075-H

PRODUCT DESCRIPTION

FEATURES

- Triangle configuration correctly distributes pressure to the tip of the shoulder.
- Suitable for adults or children.
- Adjustable hook and loop closure facilitates proper fit.

HOW TO WEAR

- Place the shoulder strap of clavicle strap over the shoulder.
- Secure the abdominal straps comfortably around the waist.
- Cross straps under armpits and place the shoulder straps through the ribbon on the back of the support. Then secure comfortably in the front to desired compression.

APPLICATION

- Poor posture
- Weak scapula muscle

What does a clavicle brace do?

The clavicle is the second-most commonly broken bone, and it's especially dangerous for kids who play high impact sports. The brace keeps their collarbone immobilized after they've had a simple break or fracture so that healing can take place! The Oppo Posture Aid or Clavicle Brace has a Triangle arrangement that accurately disperses force to the point of the shoulder. [Shop Now](#)