




## OPPO POSTURE AID / CLAVICLE BRACE

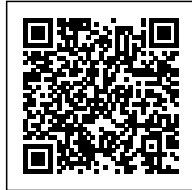

**SKU:** Opp2075-H

## PRODUCT DESCRIPTION

### FEATURES

- Triangle configuration correctly distributes pressure to the tip of the shoulder.
- Suitable for adults or children.
- Adjustable hook and loop closure facilitates proper fit.

### HOW TO WEAR

- Place the shoulder strap of clavicle strap over the shoulder.
- Secure the abdominal straps comfortably around the waist.
- Cross straps under armpits and place the shoulder straps through the ribbon on the back of the support. Then secure comfortably in the front to desired compression.

### APPLICATION

- Poor posture
- Weak scapula muscle

### What does a clavicle brace do?

The clavicle is the second-most commonly broken bone, and it's especially dangerous for kids who play high impact sports. The brace keeps their collarbone immobilized after they've had a simple break or fracture so that healing can take place! The Oppo Posture Aid or Clavicle Brace has a Triangle arrangement that accurately disperses force to the point of the shoulder. [Shop Now](#)

For further information go to **medimart.com.au** or give us a call: **1300 633 063**

## SA

### **Brighton Showroom**

343 Brighton Rd, North  
Brighton SA 5048  
08 7079 8773

[brighton@medimart.com.au](mailto:brighton@medimart.com.au)

### **Elizabeth Vale Showroom**

Playford Health Hub, 44 John Rice  
Ave, Elizabeth Vale SA 5112  
08 7079 8777

[elizabethvale@medimart.com.au](mailto:elizabethvale@medimart.com.au)

### **Reynella Showroom**

Shop 1/141 Sherriffs Rd,  
Reynella SA 5161  
08 7079 8771

[reynella@medimart.com.au](mailto:reynella@medimart.com.au)

## VIC

### **Frankston Showroom**

111 Cranbourne Rd, Frankston VIC 3199  
03 8763 3451

[frankston@medimart.com.au](mailto:frankston@medimart.com.au)

### **Monash Showroom**

23 Winterton Road, Clayton VIC 3168  
03 7503 8354

[monash@medimart.com.au](mailto:monash@medimart.com.au)

## NT

### **Darwin Showroom**

Unit 4/524 Stuart Hwy, Winnellie NT 0820  
1300 633 468

[support@medimarnt.com.au](mailto:support@medimarnt.com.au)