



OPPO STRETCHY GRIP ANKLE WRAP

\$41.00Online Only



SKU: OPP240100

PRODUCT DESCRIPTION

FEATURES

- Provides localised support and compression, and avoids unnecessary compression to areas where it is not required.
- Tension is fully adjustable to provide from light support to total compression.
- Cohesive fabric, causing no pain or irritation to the skin whilst in use, comfortable to wear and painless to remove.

HOW TO WEAR

- Place wrap on foot and let the silicone side face to skin.
- Wrap one complete turn around the arch of the foot.
- Angle upward to wrap on complete turn around the lower ankle.
- Continue wrapping in "figure eight" pattern.
- Finish and fasten the hook closure.

APPLICATION

- Weak Ankle
- Ankle Fatigue