



PELICAN SLING LEG SHEARING SLEEVES



SKU: 564

PRODUCT DESCRIPTION

Purpose & Benefits

Sling Leg Shearing Sleeves are designed to reduce the risk of skin shear and friction when lifting patients using slings, especially for those with fragile or high-risk skin. These sleeves provide a soft, protective barrier between the patient's legs and the sling straps, helping to prevent skin breakdown, bruising, and discomfort during transfers.

Ideal for use in hospitals, aged care, and home settings, the sleeves are easy to apply and remove, and are machine washable for convenient hygiene management. They can be used with most standard sling designs and are an essential comfort and safety accessory for individuals prone to pressure or shearing injuries.

When starting to lift, the Sling leg material under the thighs adjusts itself as it is being pulled up by the hoist. The material may shear against fragile skin. This could cause a wound if it is directly in contact with the patient's skin. This padded foam product allows the Sling to move inside the sleeve, without disturbing the patient's tender skin.

To Use

Slide the two Sling Leg Shearing Sleeves over the two leg sections of the Sling before putting the Sling under the patient. Ensure that the material join and product label are on the underside. The flat part of the sleeve goes next to the patient's leg. This means the leg is not resting on the seam.

Features

- Helps prevent skin shearing and bruising during hoist transfers
- Soft, cushioned fabric provides a protective barrier
- Designed to fit over sling leg straps
- Suitable for frail or high-risk skin
- Easy to put on and remove
- Machine washable for repeated use
- Compatible with most patient slings
- Ideal for hospitals, aged care, and home use

Specifications

- Product Type: Sling accessory – leg strap protector
- Material: Soft padded fabric (machine washable)
- Use: Reduces friction and shear during hoist transfers
- Compatibility: Universal – fits most sling leg straps
- Supplied As: Pair (left and right leg sleeve)

For further information go to **medimart.com.au** or give us a call: **1300 633 063**

SA

Brighton Showroom

343 Brighton Rd, North

Brighton SA 5048

08 7079 8773

brighton@medimart.com.au

Elizabeth Vale Showroom

Playford Health Hub, 44 John Rice

Ave, Elizabeth Vale SA 5112

08 7079 8777

elizabethvale@medimart.com.au

Reynella Showroom

Shop 1/141 Sherriffs Rd,

Reynella SA 5161

08 7079 8771

reynella@medimart.com.au

VIC**Frankston Showroom**

111 Cranbourne Rd, Frankston VIC 3199

03 8763 3451

frankston@medimart.com.au

Monash Showroom

23 Winterton Road, Clayton VIC 3168

03 7503 8354

monash@medimart.com.au

NT**Darwin Showroom**

Unit 4/524 Stuart Hwy, Winnellie NT 0820

1300 633 468

support@medimarnt.com.au