

RESMED AIRFIT F20 CPAP MASK



SKU: 63430-H

PRODUCT DESCRIPTION

PURPOSE & BENEFITS

The ResMed AirFit F20 Full Face Mask is one of the most advanced masks on the market and has been designed to comfortably fit a wide variety of face shapes catering to your individual needs.

KEY FEATURES

- Innovative InfinitySeal silicone cushion designed to fit everyone, tolerate misfitting and accommodate patient movement
- ResMed Mask is Comfortable and flexible frame designed to support the widest range of facial types
- Easy to use magnetic clips for quick fitting
- Clear field of vision for reading or watching TV in bed
- Women's for Her variant available for more tailored therapy

AirFit F20 Guides

- [Mask fitting video guide for the AirFit F20](#)

Produced to comfortably fit different face shapes, the AirFit F20 full face mask intends for a secure fit and comfortable seal while supporting good movement.

Buy this ResMed AirFit F20 Full Face Mask online at MediMart using Afterpay or [NDIS method](#) today!

Warning

Keep the mask magnets at a safe distance of at least 6 inches (150 mm) away from implants or medical devices that may be adversely affected by magnetic interference. This warning applies to you or anyone in close physical contact with your mask. The magnets are in the frame and lower headgear clips, with a magnetic field strength of up to 400mT. When worn, they connect to secure the mask but may inadvertently detach while asleep. Implants/medical devices, including those listed within contraindications, may be adversely affected if they change function under external magnetic fields or contain ferromagnetic materials that attract/repel to magnetic fields (some metallic implants, e.g., contact lenses with metal, dental implants, metallic cranial plates, screws, burr hole covers, and bone substitute devices). Consult your physician and manufacturer of your implant / other medical device for information on the potential adverse effects of magnetic fields.

[Click to view all ResMed affected masks and magnet locations](#)

[Click to open Patient Letter - Urgent Product Defect Correction ResMed Masks with Magnets](#)

For further information go to **medimart.com.au** or give us a call: **1300 633 063**

SA

Brighton Showroom

343 Brighton Rd, North
Brighton SA 5048
08 7079 8773

brighton@medimart.com.au

Elizabeth Vale Showroom

Playford Health Hub, 44 John Rice
Ave, Elizabeth Vale SA 5112
08 7079 8777

elizabethvale@medimart.com.au

Reynella Showroom

Shop 1/141 Sherriffs Rd,
Reynella SA 5161
08 7079 8771

reynella@medimart.com.au

VIC

Frankston Showroom

111 Cranbourne Rd, Frankston VIC 3199
03 8763 3451

frankston@medimart.com.au

Monash Showroom

23 Winterton Road, Clayton VIC 3168
03 7503 8354

monash@medimart.com.au

NT

Darwin Showroom

Unit 4/524 Stuart Hwy, Winnellie NT 0820

1300 633 468

support@medimarnt.com.au