



TECH-ASSIST SLIP GRIP

\$108.00 - \$149.00Price range: \$108.00 through

\$149.00



SKU: 15151

PRODUCT DESCRIPTION

PURPOSE & BENEFITS

The Slip-Grip (one way slide tube) is designed to slip easily in the direction of the arrow, but lock in place in the opposite direction. This allows a person to be moved up the chair or bed without fear of them slipping back down, in fact the super grip material used on the outer side of the Slip-Grip will in itself prevent someone from slipping forward again easily, therefore alleviating the need in many cases for a lap strap.

FEATURES

- Non Slip Outer Cover
- Super Slip interior
- Single direction slip, locks in reverse
- Medium wash (40°) drip dry only

INSTRUCTIONS

- 1. Place the Slip Grip on to chair with arrow pointing toward the rear of the seat
- 2. Seat the client / patient onto the Slip Grip
- 3. It is possible to reposition the client/ patient back into the chair by sliding them over the Slip Grip towards the rear of the seat
- 4. The Slip Grip is now locked in this position

DIMENSIONS

- Small 45cm x 45cm
- Large 60cm x 75cm