



TECH-ASSIST SLIP GRIP



SKU: 15151

PRODUCT DESCRIPTION

PURPOSE & BENEFITS

The Slip-Grip (one way slide tube) is designed to slip easily in the direction of the arrow, but lock in place in the opposite direction. This allows a person to be moved up the chair or bed without fear of them slipping back down, in fact the super grip material used on the outer side of the Slip-Grip will in itself prevent someone from slipping forward again easily, therefore alleviating the need in many cases for a lap strap.

FEATURES

- Non Slip Outer Cover
- Super Slip interior
- Single direction slip , locks in reverse
- Medium wash (40°) drip dry only

INSTRUCTIONS

1. Place the Slip Grip on to chair with arrow pointing toward the rear of the seat
2. Seat the client / patient onto the Slip Grip
3. It is possible to reposition the client/ patient back into the chair by sliding them over the Slip Grip towards the rear of the seat
4. The Slip Grip is now locked in this position

DIMENSIONS

- Small 45cm x 45cm
- Large 60cm x 75cm

For further information go to **medimart.com.au** or give us a call: **1300 633 063**

SA

Brighton Showroom

343 Brighton Rd, North
Brighton SA 5048
08 7079 8773

brighton@medimart.com.au

Elizabeth Vale Showroom

Playford Health Hub, 44 John Rice
Ave, Elizabeth Vale SA 5112
08 7079 8777

elizabethvale@medimart.com.au

Reynella Showroom

Shop 1/141 Sherriffs Rd,
Reynella SA 5161
08 7079 8771

reynella@medimart.com.au

VIC

Frankston Showroom

111 Cranbourne Rd, Frankston VIC 3199
03 8763 3451

frankston@medimart.com.au

Monash Showroom

23 Winterton Road, Clayton VIC 3168
03 7503 8354

monash@medimart.com.au

NT

Darwin Showroom

Unit 4/524 Stuart Hwy, Winnellie NT 0820
1300 633 468

support@medimarnt.com.au