




VULKAN INDUSTRIAL BACK SUPPORT

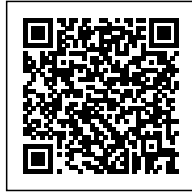

SKU: VLK-2169-H

PRODUCT DESCRIPTION

FEATURES

- Helps prevent and relieve lower back pain.
- Easy to apply and remove. Inconspicuous to wear.
- May be used for both the lumbar and lumbosacral regions.
- Four pliable support splints in the back for extra stability.

HOW TO WEAR

- Wrap support around the waist and be sure the four stays are on each side of spine.
- Adjust the support to desired compression and secure the hook and loop strap to the front of support.
- Pull the elastic reinforcement straps to tighten, then secure to the front of support.
- Bring the cross shoulder straps over the shoulders, adjust the length of the cross shoulder straps to tighten.

APPLICATION

- Low back pain
- Muscle strain/ligament sprain
- Prevent injury

SIZING

Measure around waist.

- Small 73-83cm
- Medium 83-96cm

For further information go to **medimart.com.au** or give us a call: **1300 633 063**

SA

Brighton Showroom

343 Brighton Rd, North
Brighton SA 5048
08 7079 8773

brighton@medimart.com.au

Elizabeth Vale Showroom

Playford Health Hub, 44 John Rice
Ave, Elizabeth Vale SA 5112
08 7079 8777

elizabethvale@medimart.com.au

Reynella Showroom

Shop 1/141 Sherriffs Rd,
Reynella SA 5161
08 7079 8771

reynella@medimart.com.au

VIC

Frankston Showroom

111 Cranbourne Rd, Frankston VIC 3199
03 8763 3451

frankston@medimart.com.au

Monash Showroom

23 Winterton Road, Clayton VIC 3168
03 7503 8354

monash@medimart.com.au

NT

Darwin Showroom

Unit 4/524 Stuart Hwy, Winnellie NT 0820
1300 633 468

support@medimarnt.com.au