


VULKAN OPEN KNEE SUPPORT



SKU: VLK-1124

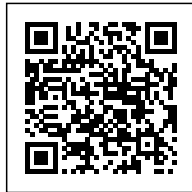

PRODUCT DESCRIPTION

KEY FEATURES

- Provides localised support and compression, and avoids unnecessary compression to areas where it is not required.
- Tension is fully adjustable to provide from light support to total compression.
- Cohesive fabric, causing no pain or irritation to the skin whilst in use, comfortable to wear and painless to remove.
- Can be used for thigh, hamstring, knee or calf.

HOW TO WEAR

- Knee - Slightly bend your knee and place wrap and let the silicone side face to skin. Begin by wrapping one complete turn around the upper calf. Continue to wrap in an upward spiral, overlapping each turn about 50%. Finish and secure the hook closure.
- Thigh - Wrap around thigh at the level required. Apply concentric overlapping strips downward. Finish and secure the hook closure.
- Shin - Start above the ankle. Apply concentric overlapping strips upward. Finish and secure the hook closure.

APPLICATION

- Knee, thigh and shin fatigue
- Muscle weakness

SIZING

Measure around knee joint.

Universal

One size fits most

Medimart has a wide range of knee guards and knee wraps with the option of paying with AfterPay.

For further information go to **medimart.com.au** or give us a call: **1300 633 063**

SA

Brighton Showroom

343 Brighton Rd, North
Brighton SA 5048
08 7079 8773

brighton@medimart.com.au

Elizabeth Vale Showroom

Playford Health Hub, 44 John Rice
Ave, Elizabeth Vale SA 5112
08 7079 8777

elizabethvale@medimart.com.au

Reynella Showroom

Shop 1/141 Sherriiffs Rd,
Reynella SA 5161
08 7079 8771

reynella@medimart.com.au

VIC

Frankston Showroom

111 Cranbourne Rd, Frankston VIC 3199
03 8763 3451

frankston@medimart.com.au

Monash Showroom

23 Winterton Road, Clayton VIC 3168
03 7503 8354

monash@medimart.com.au

NT

Darwin Showroom

Unit 4/524 Stuart Hwy, Winnellie NT 0820
1300 633 468

support@medimarnt.com.au