



VULKAN SACRO BELT

\$26.00



SKU: VLK-2262-H

PRODUCT DESCRIPTION

FEATURES

- May be used post-partum and following traumatic injury.
- Relieves instability and stress through compression of the sacroiliac joints.
- Reinforcement strap provides additional compression.
- Hook and loop closure for easy application.

HOW TO WEAR

- For use as Maternity Belt Position belt just below belly for support. Bring straps to back and fasten securely. Pull side straps to adjust tightness then fasten.
- For use as Waist Belt 1.Wrap belt around the waist. 2.Bring straps to the front and fasten. 3.Pull side straps and secure to desired compression.

APPLICATION

- Pregnancy

SIZING CHART

Measure around waist.

S	86 - 92cm
M	92 - 97cm
L	97 - 102cm
XL	102 - 110cm
XXL	110 - 118cm