




VERSAFLO3® SYSTEM

The Versaflo3 has been designed to combine high pressure care needs and comfort. Using a three layer technology, configured to create a comfortable sleep support surface while providing high pressure relief. The mattress foundation is an XP Support layer that reduces over immersion while enabling the mattress to profile. The mattress core uses adjustable cylinders that alternate air pressure to adjust the surface pressure points. ActiveXTM is used as the primary comfort layer to increase comfort and disperse pressure.

The easy to use pump, controls the pressure depending on the patient's weight. The air pressure can be manually adjusted to control the pressure and amount of air or set as a cycle for alternating pressure.

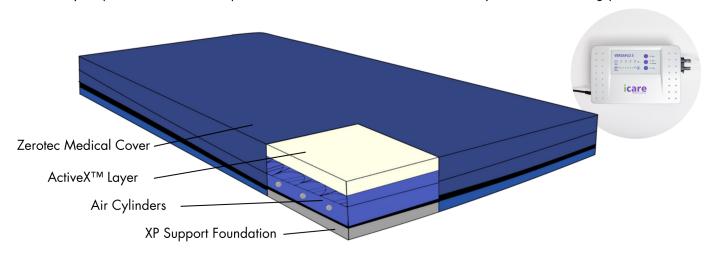

SPECIFICATIONS

Feel Rating Medium

Mattress Height 200mm

Risk Category High Risk

User Weight 20kg - 140kg

Warranty 2 Year Warranty

VERSAFLO 3 MATTRESS SYSTEM - AIR PUMP

Circulation Intervals 5min, 10min, 15min, 20min, static

Air Capacity 1.5~8L

Air Pressure 300~1200H²0 User Weight 20~140kg

VERSAFLO 3 MATTRESS SYSTEM - MATTRESS

Material Polyurethane, TPU + Fabric

Cover Waterproof
Air Pressure Level 4~5
Safe Working Load 150kg

BED CODES & SIZES

LONG SINGLE	ICVF3LS	2030 x 900mm
KING SINGLE	ICVF3KS	2030 x 1070mm

PRESSURE CARE

Risk Category High
This rating is based on our average peak pressure of <16.5mmHg.

Icare risk category ratings are based on average peak pressure, Therapist recommendation, case studies, industry comparison and the Waterlow scale. For more information refer to the Icare pressure care manual.