


10501S or L TURNING SHEET

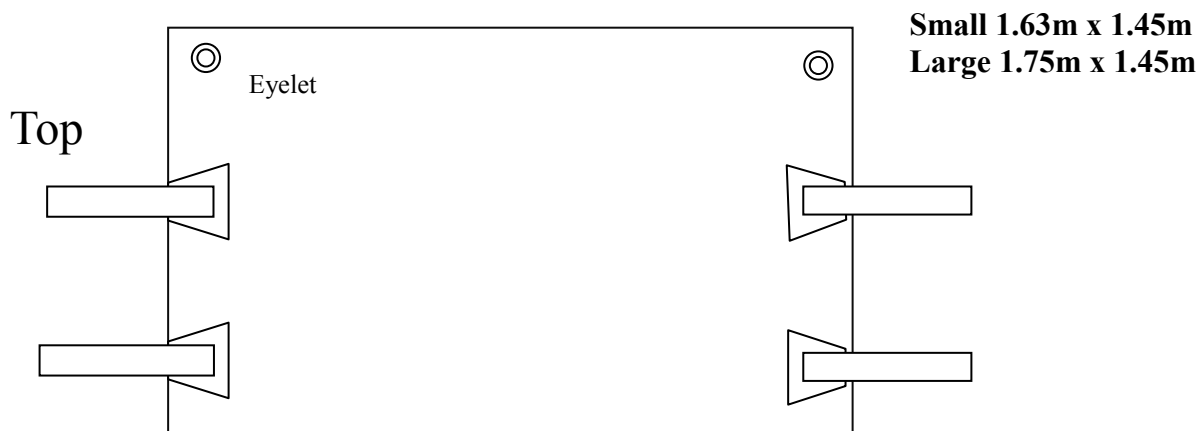

Key Features

- ◆ The Turning Sheet is used in conjunction with a mechanical hoist to turn heavy patients from side to side.
- ◆ Hoists with standard spreader bars can only be used.
- ◆ This turn sheet can be used alone or with a fitted mattress slide sheet (11701, 11705), to make turns easier, and with a sheet on top for patient comfort.
- ◆ The four side handles have three coloured loop choices to attach to the lifter hooks.
- ◆ Turn sheet can be left under patient and kept from sliding down the bed with straps threaded through eyelets and back around the bed frame.

Instructions to turn patient onto side.

1. Place Turn Sheet on bed under patient with handles protruding either side. (If Turn sheet is already under patient, ensure black holding straps, at top of bed, are un-clipped before turning).
2. Make sure bed brakes are on and raise both bed rails.
3. Patient should be in centre of bed.
4. Bring lifter to the opposite side of the bed to which you want the patient to face after turning.
5. Bed may need to be lowered depending on how high the lifter can go and which strap attachment is chosen.
6. Position patients arms and legs as for a normal log roll.
7. Attach the two side straps to lifter spreader bar ensuring same coloured strap is used
8. Start slowly raising the lifter boom and walk the lifter backwards (this will keep the patient in the middle of the bed), the patient will turn onto their side looking away from the lifter.
9. Second carer should make sure the sheet remains flat and crease free on the side the patient is turning too.
10. At completion of turn the sheet can be spread out and secured to the top of the bed using the straps threaded through the eyelets.

Washing Instructions

Machine wash Max. **60° to 70° C** with stepped or staggered temperatures for the Pre- wash & rinse cycles.
Mild soap with Ph of 7.0 recommended
Do not use bleach
Do not exceed **60° C** drying cycle with a cool down cycle.
Drip Dry preferred

IMPORTANT NOTES

- ◆ This product has **NOT** been weight tested and should **NOT** be used to completely lift the patient off the bed
- ◆ **THIS PRODUCT MUST BE ASSESSED AS APPROPRIATE FOR THE CLIENT BY AN APPROPRIATELY QUALIFIED PERSON**
- ◆ **THIS PRODUCT CAN ONLY BE USED WITH A HOIST WITH A STANDARD CROSSBAR YOKE**
- Regularly check stitching on this item. If repairs required DO NOT USE and contact supplier immediately.



IMPORTANT NOTICE TO PURCHASER: The following is made in lieu of all warranties, expressed or implied. Seller's and manufacturers only obligation shall be to replace such quantity of the product proved to be defective. Neither seller nor manufacturer shall be liable for any injury, loss or damage, direct, indirect or consequential arising out of the inability to use this product. Before using, user shall determine the suitability of the product for the intended use, and user assumes all risk and liability whatsoever in connection therewith. No statement or recommendation not contained herein shall have any force affect unless in agreement signed by officers of seller and manufacturer. All Tech-Assist Products are designed to assist nurses and patients. The patients nursing care should never be neglected, and when Tech-Assist Products are in use, continual assessment should be carried out to ensure the products are still appropriate for the patients condition. Regular maintenance inspection is necessary, and damaged items should be withdrawn from use until rectified.

Unit 2/5 Keele Place
KIDMAN PARK, SA 5025
Ph 08 82352200 Fax 08 83533200
sales@tech-assist.com.au

TECH-ASSIST PRODUCTS
www.tech-assist.com.au