



Molift EvoSling

Comfort, functionality
and safety

 **etac**[®]



Molift EvoSling

Molift has been manufacturing high quality hoists and slings for generations. Quality, comfort and safety are key considerations during product development. We offer a wide range of sling models and sizes ensuring we have a solution, whatever the users requirements.

What are the user's requirements?

For people who need help with mobility, it's important that the sling is both comfortable and safe. It must facilitate transfers and meet the user's requirements.

Comfort, function and safety

Our slings have been developed in conjunction with carers and users, in order to meet their requirements in terms of functionality, comfort and safety. The slings must be comfortable and secure, while at the same time being easy to work with for both the user and carer.

Our standard models are designed for use in the usual types of transfers and are suitable for most users. They are available in three different back heights: HighBack, MediumBack and LowBack. We know that users may have a wide variety of requirements, so we've designed additional models with specific features to meet their needs, including the EvoSling Hygiene and EvoSling Ampu.

The design of our slings ensures that they distribute weight and pressure evenly for maximum safety and comfort.

Individual testing

We recommend individual testing, and offer a wide range of different models and sizes.

Features

Flexibility

Multiple hoisting loops on both head and leg supports allow the distance to the sling bar and the user's sitting angle to be adjusted.

Colour coding

Colour coding of hoisting loops helps you find the right fit.

Padding

Padding in back and leg support distributes the weight over a larger area, making the sling more comfortable for the user.

Three different back heights

HighBack, MediumBack and LowBack to suit different needs.

Belt

EvoSling Hygiene has a belt for extra safety as well as anti-slip protection on the inside.

Labelling

Clear, practical information on a highly visible label. Washing instructions and technical information are found in a protective pocket at the back.

Guide straps

Guide straps assist with positioning during the lifting process.

Application pocket

Pockets in the back and on the leg supports make application easier.



*EvoSling
MediumBack
Padded.*

Molift EvoSling



EvoSling Comfort MediumBack



M

EvoSlings Detailed information

Recommended user weight

XS	17 – 25 kg
S	25 – 50 kg
M	45 – 95 kg
L	90 – 160 kg
XL	160 – 240 kg
XXL	230 – 300 kg

Materials

Polyester

PE-cellfoam (padded models)

Washing instructions



Washing temperature 60-85 °C / 140-185 °F

Washing at higher temperatures wears out the material faster.

Colour coding



max 300 kg | max 660 lbs




Informative, clear labelling

The large label contains all the information you will need for day-to-day use. Washing instructions, serial number and other technical information can be found behind the label in a pocket which protects against wear and tear during washing.

Periodic inspection

The sling has a label on which you can record when a periodic inspection has taken place. It can be found in the label pocket. By recording the date on which the sling is first used on the label, you can plan when the first inspection should take place.

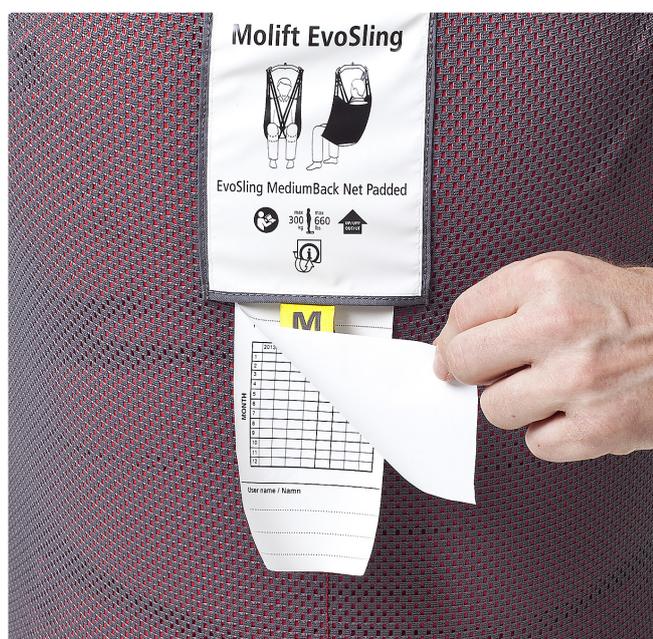

Contents

Molift EvoSling HighBack.....	7
Molift EvoSling MediumBack	8
Molift EvoSling LowBack.....	9
Molift EvoSling Hygiene	10
Molift EvoSling Ampu MediumBack.....	11
Molift EvoSling Comfort MediumBack.....	12
Molift EvoSling Shadow HighBack	13
Molift EvoSling accessories.....	14

Materials



Polyester

The standard polyester fabric is durable and hard wearing. The material is easy to maintain and has a dirt repellent surface.



Polyester net

Hard wearing and durable soft net fabric, breathes well and dries quickly. Suitable for shower and bathing situations.



Non-woven

Polypropylene non-woven. Patient specific slings when the sling is for short term use.



Molift EvoSling HighBack

For users who need a lot of support

Comfortable support

EvoSling HighBack supports the entire back and has a comfortable head support. The sling is suitable for most users, including those with poor core and head stability.

EvoSling HighBack is suitable for lifting to and from a lying position, as well as from the floor. The sling provides a comfortable reclined sitting position.

The pocket in the lower back and on the leg supports facilitates application of the sling.

Two options

Molift EvoSling HighBack comes with two options: polyester with padding in the back and leg supports and a soft polyester net with padded head and leg supports.



EvoSling HighBack Net Padded



Easier positioning

The polyester net sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.

Padded		Net Padded		Measurements		
Size		HighBack Padded	HighBack Net Padded	Size	(A) Back height	(B) Width
XS		1722810	1722710	XS	70 cm	30 cm
S		1722820	1722720	S	80 cm	35 cm
M		1722830	1722730	M	93 cm	40 cm
L		1722840	1722740	L	100 cm	47 cm
XL		1722850	1722750	XL	105 cm	54 cm
XXL		1722860	1722760	XXL	110 cm	60 cm

Molift EvoSling MediumBack

For users who have head control but need full back support

Full back support

EvoSling MediumBack is a versatile model which is suitable for the most common lifting situations. The sling provides a comfortable upright sitting position with full back support.

Users place their arms inside the sling. A pocket in the lower back and on the leg supports facilitates application of the sling. The polyester net sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.

Two options

Molift EvoSling MediumBack comes with two options: polyester with padding in the back and leg supports and a soft polyester net with padded leg supports.



Padded				Net Padded				Measurements		
Size	MediumBack Padded	MediumBack Net Padded			Size	(A) Back height	(B) Width			
XS	1723010	1723110			XS	54 cm	30 cm			
S	1723020	1723120			S	56 cm	37 cm			
M	1723030	1723130			M	60 cm	49 cm			
L	1723040	1723140			L	67 cm	57 cm			
XL	1723050	1723150			XL	70 cm	72 cm			
XXL	1723060	1723160			XXL	73 cm	81 cm			

Molift EvoSling LowBack

For active users with good core stability

Shoulder-height support

EvoSling LowBack is a versatile model which provides shoulder-height support. The sling provides a comfortable sitting position and users can place their arms inside or outside the sling.

A pocket in the lower back and pockets on the leg supports facilitate application of the sling. The polyester net sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.

Three options

Molift EvoSling LowBack comes with two options: polyester with padding in the back and leg supports and soft polyester net with padded leg supports.



Padded **Net Padded**

Size	LowBack Padded	LowBack Net Padded
XS <input type="checkbox"/>	1722310	1722510
S <input type="checkbox"/>	1722320	1722520
M <input type="checkbox"/>	1722330	1722530
L <input type="checkbox"/>	1722340	1722540
XL <input type="checkbox"/>	1722350	1722550
XXL <input type="checkbox"/>	1722360	1722560

Messurements

Size	(A) Back height	(B) Width
XS	41 cm	40 cm
S	43 cm	43 cm
M	44 cm	48 cm
L	47 cm	55 cm
XL	51 cm	62 cm
XXL	55 cm	70 cm

Molift EvoSling Hygiene

For users with good core stability and muscle tone

Easy to position

EvoSling Hygiene has a low back and a large leg support opening to facilitate dressing and undressing when using the toilet.

Application pockets


Molift EvoSling Hygiene is made of polyester and has a padded back and padded leg supports. The application pockets on the leg supports makes positioning them easier.

Extra security

EvoSling Hygiene has a belt for added security during lifting.




Padded

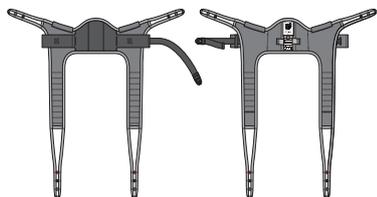

Size

Size		Padded
XS	<input type="checkbox"/>	1722210
S	<input type="checkbox"/>	1722220
M	<input type="checkbox"/>	1722230
L	<input type="checkbox"/>	1722240
XL	<input type="checkbox"/>	1722250
XXL	<input type="checkbox"/>	1722260

Measurements

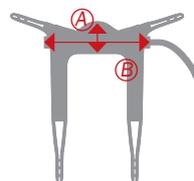

Size	(A) Back height	(B) Width
XS	29 cm	35 cm
S	33 cm	46 cm
M	33 cm	52 cm
L	33 cm	70 cm
XL	34 cm	73 cm
XXL	35 cm	76 cm

Molift EvoSling Ampu MediumBack

For users with an amputated leg or with poor muscle tone

Secure and comfortable

EvoSling Ampu MediumBack has a narrow leg support opening and a shoulder-height back. The sling's leg supports are wrapped under the legs. The leg support straps are passed through the strap guide loop to keep the leg supports together. A small leg support opening minimises the risk of the user slipping through the sling.

A pocket in the lower back and on the leg supports facilitates application of the sling. The sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.

Anti-slip for added security


EvoSling Ampu is made of polyester net and has anti-slip material on the inside under the buttocks to provide added security during lifting. The user has a comfortable reclined sitting position. The leg supports are padded for optimum comfort.



EvoSling Ampu MediumBack




Net Padded

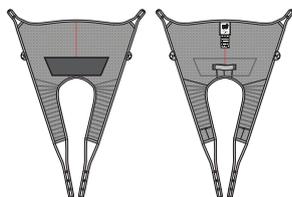

Size	MediumBack Net Padded	
XS	<input type="checkbox"/>	1723210
S	<input checked="" type="checkbox"/>	1723220
M	<input type="checkbox"/>	1723230
L	<input type="checkbox"/>	1723240
XL	<input type="checkbox"/>	1723250
XXL	<input type="checkbox"/>	1723260

Measurements

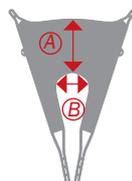

Size	(A) Back height	(B) Width
XS	29 cm	35 cm
S	33 cm	46 cm
M	33 cm	52 cm
L	33 cm	70 cm
XL	34 cm	73 cm
XXL	35 cm	76 cm

Molift EvoSling Comfort MediumBack

A full body support sling for users who need extra support

Support under the entire sitting area

EvoSling Comfort MediumBack Net is a full-body support sling without split leg support and with a shoulder-height back. The sling gives the user support under the entire sitting area and is ideal for users with joint problems or high sensitivity to pain.

The sling is positioned under the user in the lying position. It is made of polyester net, which provides good ventilation and can be left in the wheelchair underneath the user.

Adjustable sitting angle

Multiple hoisting loops on both the head and leg supports allow the sitting angle to be adjusted.

The sling has a red seam in the middle of the back to help ensure that it is positioned symmetrically.




EvoSling Comfort Medium Back



Guide straps


The guide straps are built into the side straps to ensure that the sling is comfortable.

Net

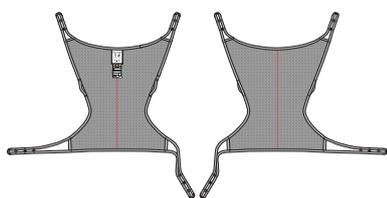

Size	MediumBack Net	
XS <input type="checkbox"/>	1722610	
S <input type="checkbox"/>	1722620	
M <input type="checkbox"/>	1722630	
L <input type="checkbox"/>	1722640	
XL <input type="checkbox"/>	1722650	
XXL <input type="checkbox"/>	1722660	

Measurements

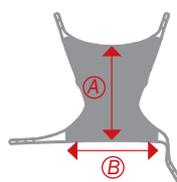

Size	(A) Back height	(B) Width
XS	77 cm	85 cm
S	90 cm	86 cm
M	102 cm	94 cm
L	115 cm	113 cm
XL	118 cm	120 cm
XXL	120 cm	126 cm

Molift EvoSling Shadow HighBack

A flexible sling that adapts to the contours of the body

May be left in place in a wheelchair or seat

EvoSling Shadow HighBack is a high sling with a split leg support, ensuring comfort and flexibility. This model suits users with need for extra support, as it adapts to the contours of the body.

Shadow Highback works in many hoisting situations, but is particularly suitable for transfers to wheel-chairs, including those with moulded seats, when the sling needs to be left in place under the user.

The EvoSling Shadow has six straps with multiple hoisting loops, giving it flexibility. The sitting angle and head support can be adjusted to the individual user, and the user's arms may be placed inside or outside the sling.

Follows the contours of the body

The EvoSling Shadow HighBack is made from a flexible polyester mesh which is permeable to moisture and heat. This model has no pockets or padding and has an extra-thin edge strip. The EvoSling Shadow can be left in place in the wheelchair or seat after the transfer.




The head support is adjustable to three different positions. The larger sizes, M-XL have a curved head support.




EvoSling Shadow HighBack

Net

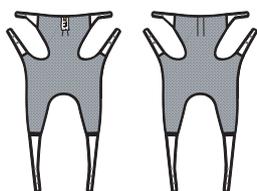

Measurements

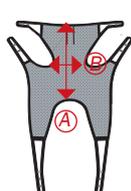

Size	HighBack
XXS 	1723900
XS 	1723910
S 	1723920
M 	1723930
L 	1723940
XL 	1723950

Size	(A) Back height	(B) Width
XS	65 cm	28 cm
S	71 cm	30 cm
M	78 cm	31 cm
L	90 cm	33 cm
XL	98 cm	40 cm
XXL	102 cm	51 cm

Molift EvoSling Accessories




Molift EvoSling Support

Accessories for slings without head support

The head support is attached to the sling and its length can be adjusted as required using the plastic buckle. It gives the user comfortable support for the head and is a useful addition when the user requires more support.

Product name	Item no.
Molift EvoSling Support	1723400

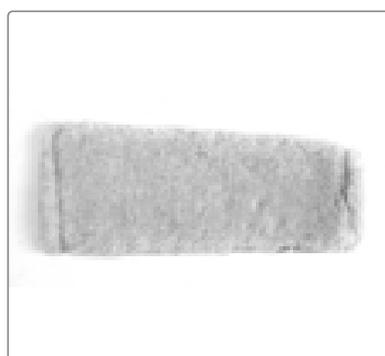

Molift EvoSling Soft Padding

For pressure-sensitive users

EvoSling Soft Padding consists of pressure-relieving padded sleeves which are slipped over the leg parts of slings with split-leg support.

Product name (sold in pairs)	Size	Item no.
Molift EvoSling Soft Padding	XS-M	1723710
Molift EvoSling Soft Padding	L-XXL	1723810
Molift EvoSling Hygiene Soft Padding	XS-XXL	1723740



Molift EvoSling Extension Loop

Straps for easier adjustment

Used to extend the hoisting straps or to keep the leg supports together on slings with split-leg support.

Product name	Size	Item no.
Molift EvoSling Extension loop	16 (+9) cm	1723510
Molift EvoSling Extension loop	25 (+18) cm	1723520



Molift Strap Marker

For marking a selected hoisting loop for head or leg support hoisting straps. Mark the hoisting loops selected during individual testing of the sling.

Product name (sold in pairs)	Item no.
Etac Molift Strap Marker	1723600

Molift EvoSling Accessories



Molift EvoSling FlexiStrap

Avoid heavy lifting

The Molift EvoSling FlexiStrap is used to easily and comfortably lift a leg or arm during care or handling. The hoisting strap is made of padded polyester and comes in two widths. There is a pocket on one side of the strap to facilitate positioning.

Product name (sold in pairs)	Size	Item no.
Molift EvoSling FlexiStrap, standard	15 x 95 cm	1723310
Molift EvoSling FlexiStrap, wide	25 x 95 cm	1723330




Immedia SlingOn

When the user can't be turned onto their side

EvoSling Soft Padding consists of pressure-relieving padded sleeves which are slipped over the leg parts of slings with split-leg support.

Product name (sold in pairs)	Size	Item no.
Immedia SlingOn - Small	80 x 100 cm	IM437/S
Immedia SlingOn - Medium	80 x 170 cm	IM438
Immedia SlingOn - Large	120 x 150 cm	IM437

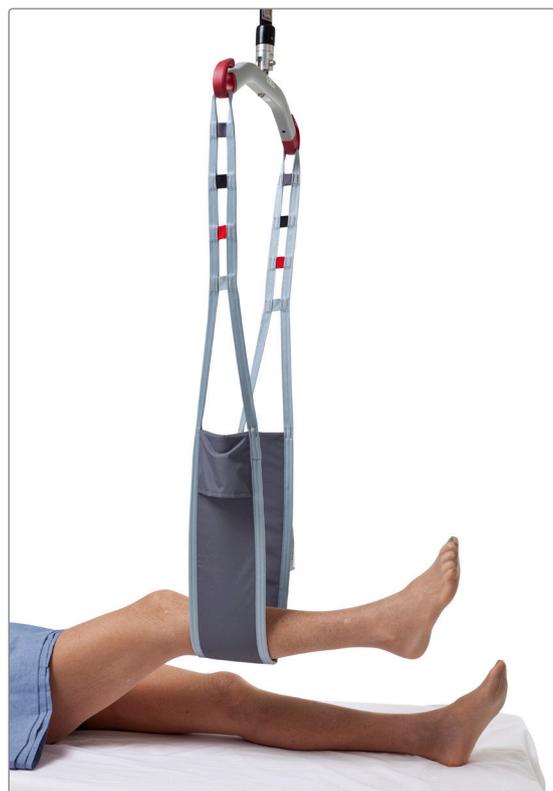



June 10, 2021

Etac is a world-leading developer and provider of ergonomic assistive devices and patient handling equipment. Our heart lies in the solutions that optimise quality of life for the individual, their family and caregivers.

For the latest news and continuously updated product information – visit: www.etac.com

Etac AB
Kista Science Tower
SE-164 51 Kista, Sweden
Tel +46 371 - 58 73 30
info@etac.se www.etac.com


Creating Possibilities