

APPROVED CLEANING PRODUCTS FOR MATTRESS COVERS, UPHOLSTERY TOPS AND SHOWER TROLLEY TUB FABRICS:

| APPROVED CLEANING PRODUCTS | MAXIMUM CONCENTRATION |
|--|--|
| Actichlor Plus | Use according to the manufacturers specifications. |
| Actimax | Use according to the manufacturers specifications. |
| ALCO SANNI | Use according to the manufacturers specifications. |
| ALCO-DC-7 | Use according to the manufacturers specifications. |
| Alcohol wipes 70% Isopropyl Alcohol | Use according to the manufacturers specifications. |
| Arjo Disinfectant | Use according to the manufacturers specifications. |
| Azo max multi surface wipes | Use according to the manufacturers specifications. |
| Azo wipe hard surface wipes | Use according to the manufacturers specifications. |
| Chlor-Clean Detergent | Use according to the manufacturers specifications. |
| Cidal, disinfectant/cleaner | Use according to the manufacturers specifications. |
| Clinell Detergent Wipes | Use according to the manufacturers specifications. |
| Clinell Hand & Surface Sanitizer Disinfectant | |
| Clinell Sporidical Wipes (GlobalMedics) | |
| Clinell Universal Sanitising Wipes | |
| Dettol | Use according to the manufacturers specifications. |
| Ecopine | 5% Ecopine (1 part Ecopine to 20 parts water) |
| Hygena wash, shampoo and body wash | Use according to the manufacturers specifications. |
| JUMBO Liv-Wipes | Use according to the manufacturers specifications. |
| Leggeside HB (Legge Systems) | Use according to the manufacturers specifications. |
| Mediwipes Disinfection (Sulco Limited) | Use according to the manufacturers specifications. |
| Mikrocid® universal liquid | Use according to the manufacturers specifications. |
| Neutral Detergent Wipes (Reynard Health Supplies) | Use according to the manufacturers specifications. |
| NON ACID bathroom cleaner | Use according to the manufacturers specifications. |
| PAL Detergent Wipes | Use according to the manufacturers specifications. |
| PAL Disinfectant Wipes (GlobalMedics) | |
| PCS 5000 wipes | Use according to the manufacturers specifications. |
| Perform® sterile PAA | Use according to the manufacturers specifications. |
| PureGreen 24 (Pure Green) | Use according to the manufacturers specifications. |
| Sanicare TBX | Use according to the manufacturers specifications. |
| Sanicare Disinfectant Wipes (Buckeye International) | |
| Sporeclear | Use according to the manufacturers specifications. |
| Spring Clean (Eco Lab) | 5% Spring Clean (1 part Spring Clean to 20 parts of water) |
| Sterri-Matt Detergent | Use according to the manufacturers specifications. |
| Super Sani Cloth -Germicidal Disposable Wipes (PDI) | Use according to the manufacturers specifications. |
| Surface Disinfecting Wipes | Use according to the manufacturers specifications. |
| Neutral Detergent Wipes (Reynard Health Supplies) | |
| S-7 Xtra Wipes | Use according to the manufacturers specifications. |
| Terminator One Step Disinfectant (Buckeye International) | Use according to the manufacturers specifications. |
| Therasol | Use according to the manufacturers specifications. |
| TPH protect | Use according to the manufacturers specifications. |
| Triad (by Diversey) | Use according to the manufacturers specifications. |
| Trigene (Bio Strategy) | Use according to the manufacturers specifications. |
| Tristel Fuse | Use according to the manufacturers specifications. |
| Tuffie 5 Cleaning Wipes (Vernacare) | Use according to the manufacturers specifications. |
| Turbo cleaner, hard surface cleaner/detergent | Use according to the manufacturers specifications. |
| Viraclean Benzalkonium Chloride | Use according to the manufacturers specifications. |
| Virkon (Dupon) | 5% Virkon (1 part Virkon to 20 parts water) |

Example - Use 5% Virkon (1 part Virkon to 20 parts of water)

Use 25ml per 0.5 litres of water. Use 50ml per 1 litre of water.
Use 100ml per 2 litres of water. Use 150ml per 3 litres of water.

Howard Wright Limited has a policy of continuous product improvement and reserves the right to change product specifications without notice. Howard Wright Limited is certified to ISO13485:2003.



CLEANING INSTRUCTIONS

GENERAL CLEANING INSTRUCTIONS (ALL SURFACES)

To wash:

- Wipe with a sponge or soft cloth wetted with warm water containing soap or mild detergent.
- The mattress cover can be machine washed (95°C max).

To dry:

- Wipe with a dry sponge or soft cloth.
- Place mattress cover in a dry environment (if further drying is required).
- Cover can be tumble dried at a temperature below 50°C but must be removed from the drier immediately when dry and not be left in the machine. Interrupting the drying cycle occasionally to redistribute the covers may reduce drying time (only bielastic mattress covers).

To disinfect:

- Wash the surface.
- Wipe surface with a sponge or soft cloth wetted with an approved cleaning product.
- Wipe off residue with water using a soft cloth or sponge.
- Dry.

BLEACH CLEANING INSTRUCTIONS

If none of the approved cleaning products provide the disinfectant level required then use a solution of 5.25% Sodium Hypochlorite diluted to less than 1 part bleach to 100 parts water (volume to volume).

Bleach products are corrosive and degrading in nature and may cause damage to your product. If bleach is used, measures must be taken to ensure the product is rinsed thoroughly with clean water and thoroughly dried. Failure to rinse and dry the product will leave a corrosive residue on the surface possibly causing corrosion.

Failure to follow the above directions may void the product's warranty.